



Finished design from exocad whatever way you want using My Pathfinder scanbody

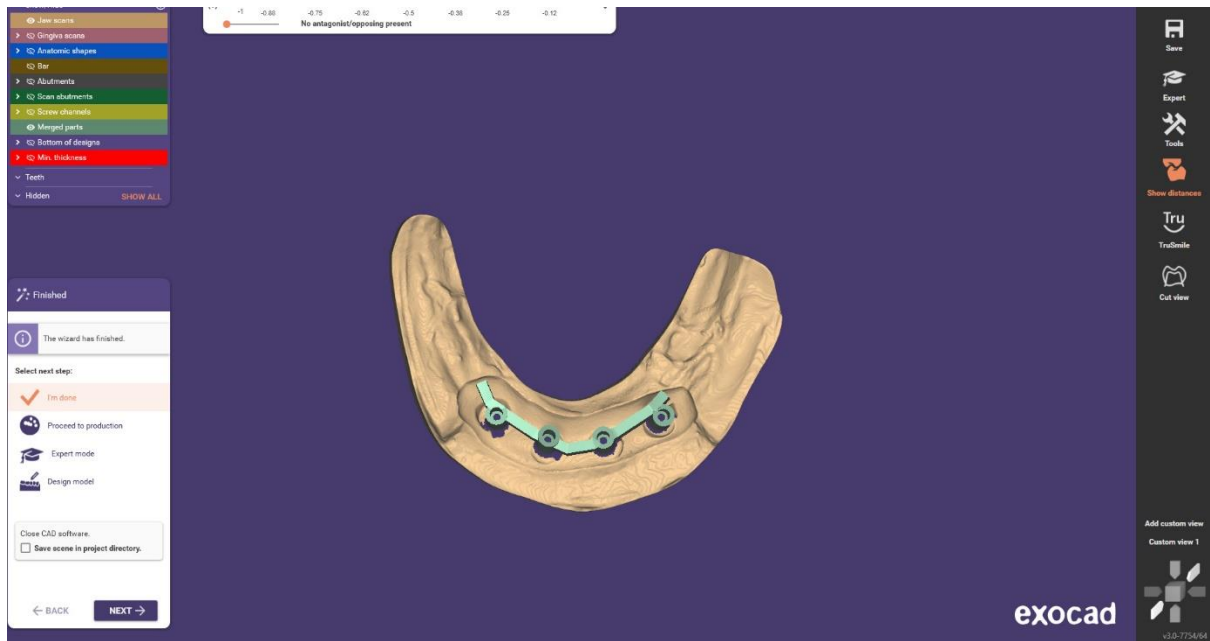

Click on I am done

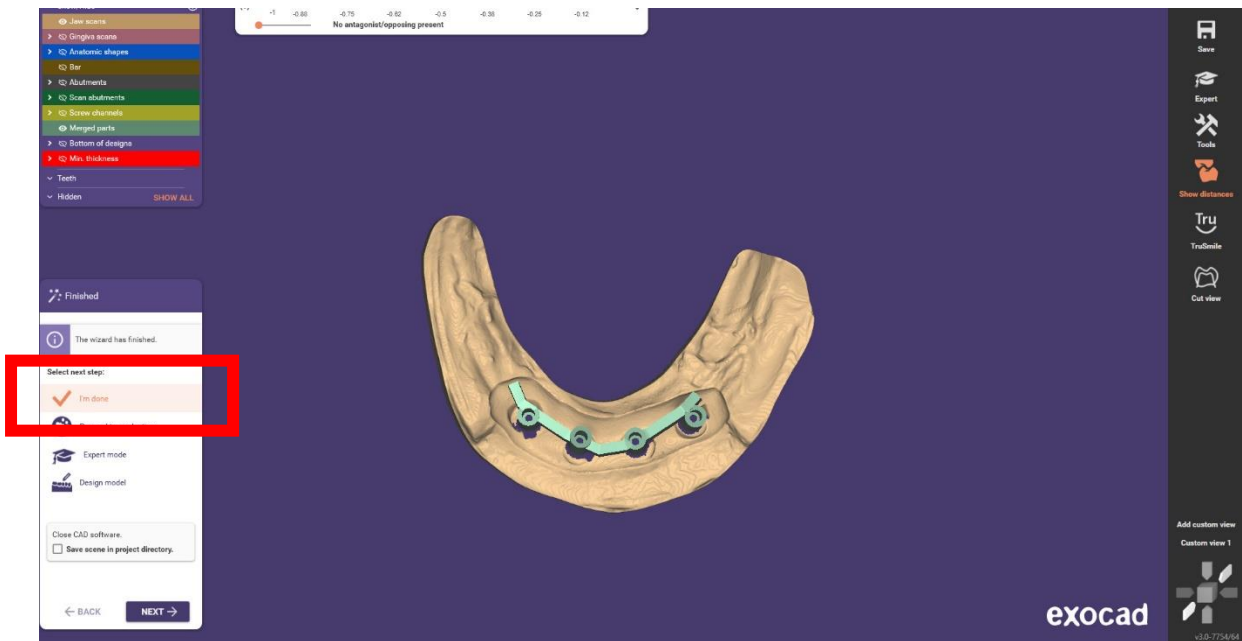

Come to main page of same project

Project

Select the next action from the toolbar on the right.

10-05-2024 18:31 Due Case ID 01095-001

Client: Default ID: 01095

Name: .bar trial MUA library

Technician: Default ID: 001

Notes

Type notes here...

Data and status

2024-05-10_01095-00... 2024-05-10_01095-00... 2024-05-10_01095-00... 2024-05-10_01095-00...

Open in explorer

Indication and Materials

Patient Multi-die

Bar pillar
Bar segment

Tooth shade Scan mode
One stone model only

Actions

- Design
- Model Creator
- Manufacture
- dentalshare
- Copy
- Print
- Settings...
- About...

exocad

Click on open to page of same explorer

Project

Select the next action from the toolbar on the right.

10-05-2024 18:31 Due Case ID 01095-001

Client: Default ID: 01095

Name: .bar trial MUA library

Technician: Default ID: 001

Notes

Type notes here...

Data and status

2024-05-10_01095-00... 2024-05-10_01095-00... 2024-05-10_01095-00... 2024-05-10_01095-00...

Open in explorer

Indication and Materials

Patient Multi-die

Bar pillar
Bar segment


Tooth shade Scan mode
One stone model only

Actions


- Design
- Model Creator
- Manufacture
- dentalshare
- Copy
- Print
- Settings...
- About...

exocad

Get location of your project save

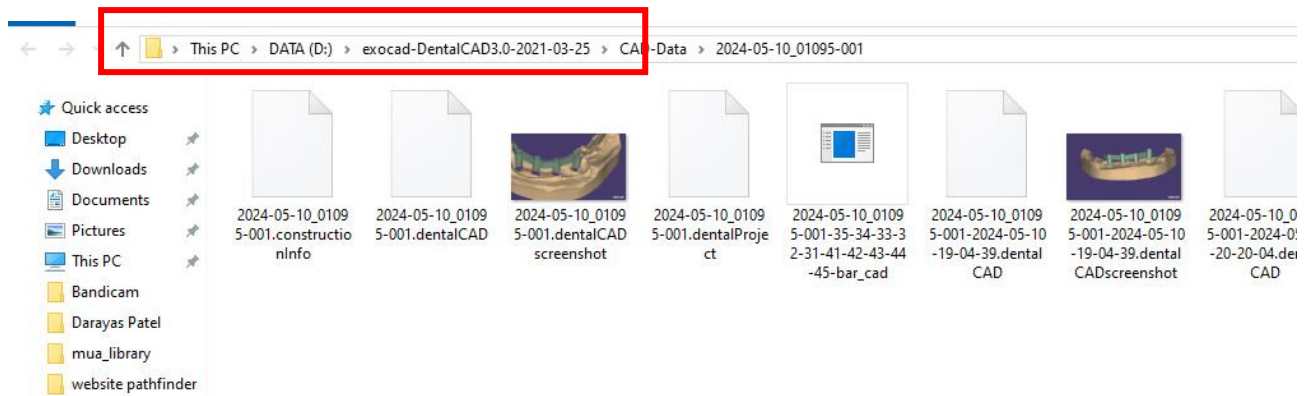

Click here to reach folder of your project

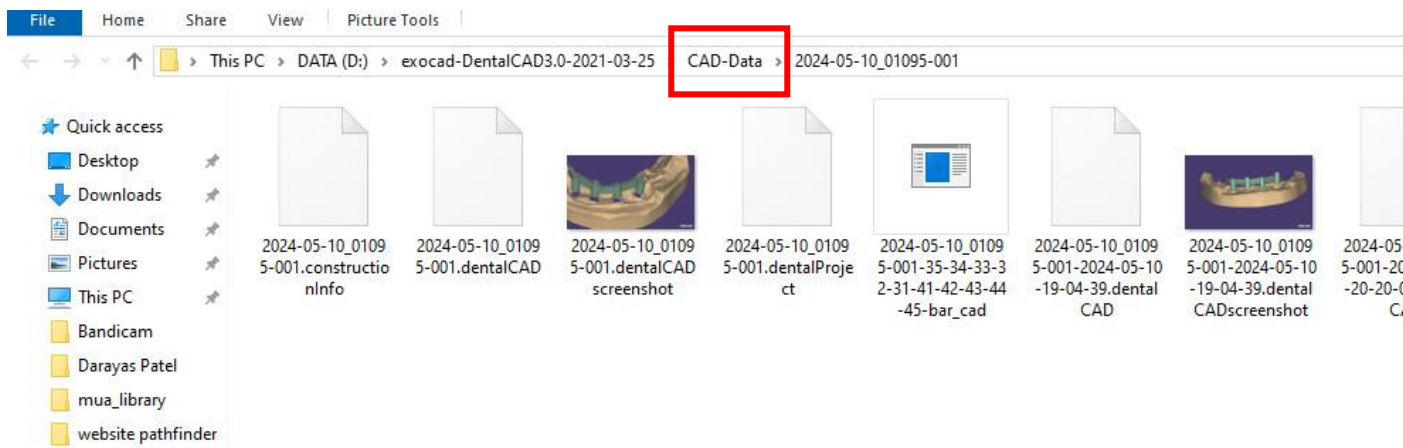

Make zip file of your project

2024-03-07_00001-001	07-03-2024 18:05	File folder	
2024-03-11_00001-001	11-03-2024 11:06	File folder	
2024-03-20_01095-001	20-03-2024 13:05	File folder	
2024-03-21_01095-001	21-03-2024 19:38	File folder	
2024-03-21_01095-002	21-03-2024 20:14	File folder	
2024-03-21_01095-003	21-03-2024 20:32	File folder	
2024-03-21_01095-004	21-03-2024 20:36	File folder	
2024-03-23_01095-001	27-03-2024 09:12	File folder	
2024-03-27_01095-001	27-03-2024 15:17	File folder	
2024-03-27_01095-002	31-03-2024 06:49	File folder	
2024-04-03_01095-001	03-04-2024 07:50	File folder	
2024-04-10_01095-001	10-04-2024 16:56	File folder	
2024-04-12_01095-001	15-04-2024 19:37	File folder	
2024-04-12_01095-002	12-04-2024 19:09	File folder	
2024-05-10_01095-001	11-05-2024 16:10	File folder	
2024-05-10_01095-001	15-05-2024 07:06	WinRAR archive	1,33,286 KB
DentalIPP_V3.sqlite	15-05-2024 06:47	SQLITE File	8,786 KB

Upload this zip file to website upload button

Enjoy My Pathfinder